

Jack set up instructions

Product Name: Jack Multi Adaptor

Product Code: RG 4

Description: The 'Jack' universal adapter enables you to attach more than one item to a nurse call socket such as a Call button. It also includes a safety feature which alerts staff when a call lead or any other accessory is removed.

Connection Points

- a. Mono/Stereo switch - Switch according to the configuration of your system - see list below. If you are unsure or your system does not appear on the list, please contact your nearest Ramblegard distributor for further details.

Intercall – Mono


Quantec – Stereo

Nursecall 800 – Stereo

C-TEC – Stereo

- b. Sensor Mat Socket.
- If its a Bedgard or Seatgard attach the coiled lead, connecting to the monitor, in here.
- If its a Floorgard attach the USB lead in here.
- c. Call Cord Socket Please ensure the 'Port' switch is at the on position when an accessory is plugged into the Call Cord Socket.
- d. On/off switch for Call Cord socket.

The adapter is now configured for your system.

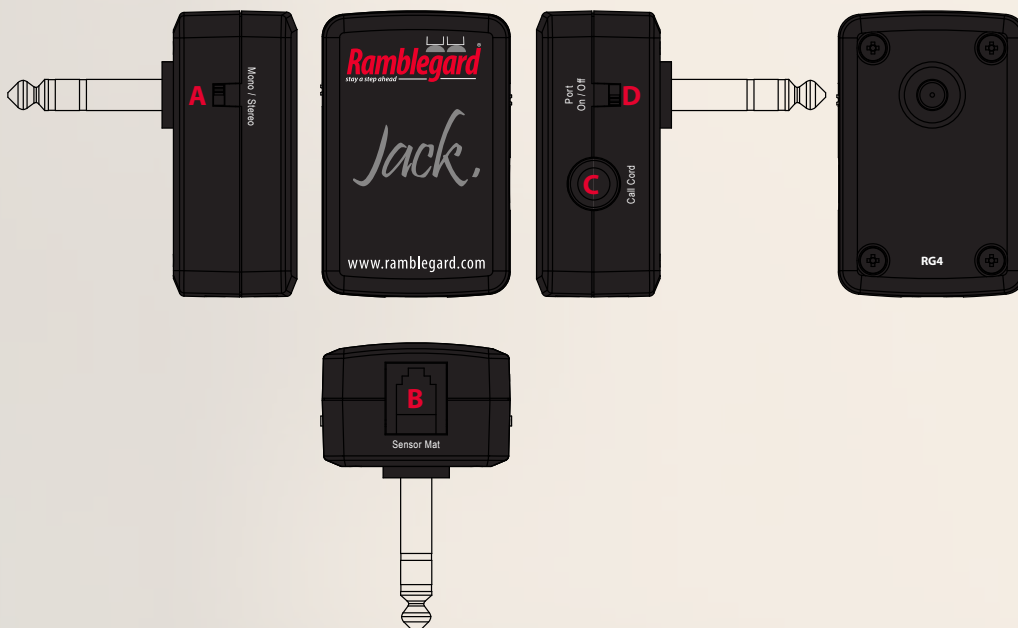

NOTE: for illustrative purposes only

Ramblegard
stay a step ahead